

What's new

The new Options footwear catalogue has been developed in response to feedback from orthotists and patients alike. Our range includes new designs, an updated colours and materials range and new last shapes to offer greater choice.



Improvements include:

- + New shoe styles added to mens and womens range
- + New last shapes developed for mens and womens range
- + New childrens range
- + 'Classic' last shape updated
- + New toe shapes for additional cosmetic acceptance
- + Improved measuring guide
- + Updated colour range
- + Updated materials range
- + Improvements in manufacturing and quality processes
- + Improved checks for last accuracy
- + Investment in new technology to improve finish including heel counters and toe puffs
- + Change in insoles provided to 3mm A40 EVA, 3mm A30 EVA and Poron top layer with leather cover

Colours and materials

The colours and materials within this range have been carefully selected to maintain our core traditional styles, but also to include more fashionable colours, designs and textures.



CALF

This fine grain leather is durable and soft.

SOFTEE

Softee leather is smooth and malleable.

TEXTURED SOFTEE

A softee leather with a patterned texture embossed onto it.

NUBUCK

Attractive finish with a superb velvet-like surface.

PATENT

High gloss finish that can be wiped clean with a damp cloth.

COLOURS AND MATERIALS RANGE

The following colours come with blind eyelets, bottle green, milk, aubergine and cobalt.

CALF AND SOFTEE



^{*} These colours come as white contrast only. This includes white fastening, laces, eyelets, welt, stitching and sole unit.

TEXTURED SOFTEE



NUBUCK



PATENT





There are three new choices of toe shapes; chisel toe and rounded toe for ladies and pointed toe for mens.

The original classic last has also been updated taking into account orthotist feedback.

For all new last shapes, two sizes in length for mens and one and a half for ladies have been added to the end of the last for cosmetic effect.

The measuring process will remain the same, but please note that shoes with additional cosmetic length will appear longer when they are delivered.

All shoes are available in all last options. Please consider toe shape on the style when ordering.



LADIES FOOTWEAR RANGE

LINSEY **VELCRO**



Pictured



TAUPE



PYRAMID

Pictured







Comes with separate sole and heel as standard.

NEW

ALICE

LAY-ON VELCRO SANDAL



Pictured







Not pictured to default specification, comes with separate sole and heel as standard.

PRIMROSE

T-BAR SANDAL



Pictured







PYRAMID

Pictured





BLUE







LADIES FOOTWEAR RANGE

NEW

KACEY

MARY JANE SHOE



Pictured







PYRAMID

Pictured



ROUNDED







CHIFFON

CLASSIC LADIES DERBY



Pictured







Pictured









CHALLIS

MARY JANE SHOE



Pictured







Pictured











SORREL STITCHED QUARTER PATTERN



Pictured



CLASSIC



PYRAMID

Pictured



ROUNDED WILLOW





Comes with separate sole and heel as standard.

POPLIN

LOW OPENING WHOLE CUT



Pictured







Not pictured to default specification, comes with separate sole and heel as standard.

ROSA

LADIES BROGUE



Pictured







PYRAMID

Pictured







PATENT







LADIES FOOTWEAR RANGE

ORCHID

DAISY PUNCH VAMP DERBY



Pictured



CLASSIC



PYRAMID

Pictured



ROUNDED SCHOOL GREY







PEONY

SUEDE VAMP DERBY AND QUARTERS



Pictured







Only available in the following colours:













GREEN

BARBIE PINK

MADEIRA

CHAMBRAY

LOW OPENING DERBY



CLASSIC







ANN WHOLE CUT SHOE



Pictured







ROUNDED AUBERGINE PYRAMID

NEW

AMY WHOLE CUT TRAINER



Pictured









CHANTILLY

WHOLE CUT TRAINER









LADIES FOOTWEAR RANGE



AIREDALE

LACE FASTENING COURT SHOE









NEW

MORPETH

5 EYELET CHUKKA BOOT



Pictured







Only available in:







NUBUCK

NUBUCK

Comes with leather stitch mock welt and lightweight commando sole unit.

ZINNIA

DERBY BOOT









Pictured



CLASSIC







PYRAMID

LADIES FOOTWEAR RANGE

STOCK DEFAULTS, STOCK OPTIONS AND MODULAR OPTIONS



HOUSE SHOE

INDOOR USE SHOE WITH VELCRO STRAP



Pictured







PYRAMID

Available with red, black or blue pipe welt.

			Fa	stenin	gs			Oper	nings		Sole & Heel			
Style	Page Number	Lace	1 Strap	2 Strap	3 Strap	Buckle	Normal	Low 12mm	Extra Low 18mm	Extended 25mm	Separate Sole & Heel	Rocker	Through Wedge	Semi Wedge
Linsey	7													
Alice	7													
Primrose	7													
Kacey	8													
Chiffon	8		•											
Challis	8													
Sorrel	9			•										
Poplin	9			•										
Rosa	9													
Orchid	10													
Peony	10		•											
Chambray	10	•												
Ann	11			•										
Amy	11			•										
Chantilly	11													
Airedale	12		•											
Morpeth	12			•										
Zinnia	12			•										
House Shoe	13													

MENS FOOTWEAR RANGE

ALNWICK

CLASSIC DERBY



Pictured







PYRAMID

Pictured









MARSDEN

LOW OPENING DERBY









PYRAMID

Not pictured to default specification, comes with separate sole and heel as standard.

LANGLEY

DERBY BROGUE









PYRAM

FULWELL STITCHED VAMP DERBY









PYRAMID

Pictured



ACORN

BLACK





NEW

AUCKLAND





Pictured







NEW

BERWICK

DERBY BROGUE



Pictured







BLACK

Note this style is pictured on the chisel last. This is the only mens shoe with this last option.

This is a restricted access shoe.

MENS FOOTWEAR RANGE

ROTHBURN

OPFN-TO-TOF



Pictured







Not pictured to default specification, comes with separate sole and heel as standard.

NEW

KIRKSTALL

DERBY BROGUE



Pictured







PYRAMID

NEW

DURHAM

LOW ANKLE BOOT



Pictured







Kicker sole unit is not available for narrow last A and is only available in black or white, please consider your upper colour when ordering.

Please note leg height comes as standard 100mm. Please see page 33 for variances on leg circ.

This is a restricted access boot.

BELFORD

DERBY BOOT



Pictured







WHITBURN

OPEN-TO-TOE BOOT



Pictured







NEW

MORPETH

5 EYELET CHUKKA BOOT



Pictured







Only available in:









Comes with leather stitch mock welt and lightweight commando sole unit.

MENS FOOTWEAR RANGE



JARROW

WHOLE CUT MENS TRAINER



Pictured







NEW

DELAVAL

OPEN-TO-TOE MENS TRAINER



Pictured









WALTON

WHOLE CUT TRAINER









MENS FOOTWEAR RANGE

STOCK DEFAULTS, STOCK OPTIONS AND MODULAR OPTIONS



HOUSE SHOE

INDOOR USE SHOE WITH VELCRO STRAP



Pictured







PYRAMID

Available with red, black or blue pipe welt.

		Fastenings						Openings			Sole & Heel			
Style	Page Number	Lace	1Strap	2 Strap	3 Strap	Buckle	Normal	Low 12mm	Extra Low 18mm	Extended 25mm	Separate Sole & Heel	Rocker	Through Wedge	Semi Wedge
Alnwick	14													
Marsden	14			•										
Langley*	14													
Fulwell	15			•										
Auckland	15													
Berwick	15													
Rothburn	16			•										
Kirkstall	16													
Durham	16													
Belford	17													
Whitburn	17													
Morpeth	17													
Jarrow	18			•										
Delaval	18			•										
Walton	18			•										
House Shoe	19													

[▲] Stock Default

Stock Option

Modular Option

^{*} Please note when ordering the low 12mm opening option on the Langley, elements of the pattern will be removed.

CHILDRENS FOOTWEAR RANGE

Our childrens range of shoes are made from heavy calf leather or nubuck in the colours illustrated.

All childrens footwear come with a 3mm Poron inlay as standard.

This is a stock range only and all orders placed must go straight to a completion.

Please note mustard is only available in nubuck.



TIGER

VELCRO OR LACE FASTENING + REHABILITATION + STABILITY



Available



BLACK







MUSTARD

NEW

VELCRO OR LACE FASTENING













MUSTARD





PANTHER

VELCRO OR LACE FASTENING + REHABILITATION + STABILITY















Kicker sole unit available in black and white

CHILDRENS FOOTWEAR SIZES

Note: Size 25 and below will come with two straps as standard and size 26 and above will come with three straps.

Si	ze	Narrow	Fitting	Standar	d Fitting	Wide	Fitting
UK	EU	Circ (cm)	Width (mm)	Circ (cm)	Width (mm)	Circ (cm)	Width (mm)
0.5	15	11.35	43	12.35	47	13.35	51
1	16	11.75	45	12.75	48	13.75	52
1.5	17	12.15	46	13.15	50	14.15	54
2.5	18	12.55	48	13.55	52	14.55	56
3	19	12.95	50	13.95	54	14.95	57
4	20	13.35	51	14.35	55	15.35	59
5	21	13.75	53	14.75	57	15.75	60
5.5	22	14.15	55	15.15	58	16.15	62
6	23	14.55	57	15.55	60	16.55	64
7	24	14.95	58	15.95	61	16.95	65
8	25	15.35	59	16.35	63	17.35	67
8.5	26	15.75	61	16.75	65	17.75	68
9.5	27	16.15	63	17.15	66	18.15	70
10	28	16.55	64	17.55	68	18.55	71
11	29	16.95	65	17.95	69	18.95	73
12	30	17.30	66	18.30	71	19.30	74
12.5	31	17.65	68	18.65	72	19.65	76
13.5	32	18.00	69	19.00	73	20.00	78
1	33	18.40	71	19.40	75	20.40	79
2	34	18.80	72	19.80	76	20.80	81
2.5	35	19.20	74	20.20	78	21.20	82
3.5	36	19.60	75	20.60	79	21.60	83
4	37	20.00	77	21.00	81	22.00	85
5	38	20.40	78	21.40	82	22.40	86
6	39	20.80	80	21.80	84	22.80	88
6.5	40	21.25	81	22.25	86	23.25	89

HEEL STIFFENERS

Rehabilitation heel stiffeners have a firm heel counter and are extended medially and laterally to behind the met heads.

Stability heel stiffeners are padded with additional height to below the collar and are extended forward on both sides to the mets.

Standard heel stiffeners are also available and are the default option.

Sole Units

Each style of shoe in this catalogue is pictured with the most popular sole material. Lightweight commando, mens trainer and kicker style sole units are only recommended by Options on certain styles (as featured in the catalogue).

Some sole units featured are only available at an additional cost, this includes lightweight commando, kicker style, PU non-moulded and leather sole units.

For further information on this, please talk to your Key Account Manager or contact us via the details at the back of this catalogue.

Ladies shoes come with pipe welt as standard, except lightweight commando, leather and kicker style soles.

Mens shoes come with standard mock welt, except lightweight commando, kicker style and mens trainer soles.

^{*} Please note: The kicker style sole units are not suitable for narrow last A



Technical Information

FEATURES

- + 9mm insole allowance
- + Lace or velcro alternatives
- + Diabetic option available, which includes plush lining, rim toe puff, padded collar and full quarters

STOCK/MODULAR/MADE TO MEASURE FOOTWEAR

The chart below gives guidance as to when footwear is stock (made to a pre-determined size and specification), modular (made with limited individual modifications) and made to measure – M2M.

	Stock	Modular	Made to Measure
Colour	Any in catalogue	Any in catalogue	Any
Material	Any in catalogue	Any in catalogue	Any
Style	Any in catalogue	Any in catalogue	Any
Size	As catalogue including 1/2 sizes	As catalogue including 1/2 sizes	Bespoke
Fitting	As catalogue including A, B and C last	As catalogue including A, B and C last	Bespoke
Last Adaptation	None	See last adaptation page (pg. 24)	Bespoke
Upper Modification	None	See upper modification page (pg. 24)	Bespoke
Sole Adaptation	Any	Any	Any
Trial Fit Available	No	Yes	Yes
Toe Puff	Standard/Rim	Standard/Rim	Any
Insole Allowance	9mm	See last adaptation page (pg.24), default 9mm	Any
Diabetic Specification	Yes	Yes	Yes

Please note: Made to Measure footwear is not included in the scope of this catalogue.

For stock footwear no last adaptations are possible.

For modular footwear, up to three adaptations per shoe can be made, each up to 12mm with the total last adaptation not exceeding 12mm over one point.

The default is to maintain tread width. Length can be increased by one size.

Lasts cannot be made smaller.

- + Length up to +1 size
- + Toe box depth up to +12mm
- + Medial joint width up to +12mm
- + Lateral joint width up to +12mm
- + Throat depth by up to +12mm
- + Instep depth by up to +12mm
- + Through depth by up to +12mm

Please record any depth increase as depth in mm, not circumference.

The formula is to half the circumference increase required to calculate the depth increase.

Last Adaptation Upper Modification


For stock footwear no upper modification is possible.

For modular footwear, the following modifications are possible with the standard upper:


- + Raise or lower back collar height at the back by 9mm
- + Raise or lower medial or lateral collar height by 4mm
- + Alter straps to any length

A modular upper can be customised at an additional cost as follows:

- + Raise or lower back collar height by more than 9mm
- + Raise or lower medial or lateral collar height by more than 4mm
- + Lower facing points by up to 25mm to make a lower opening as detailed in charts on pages 13 and 19

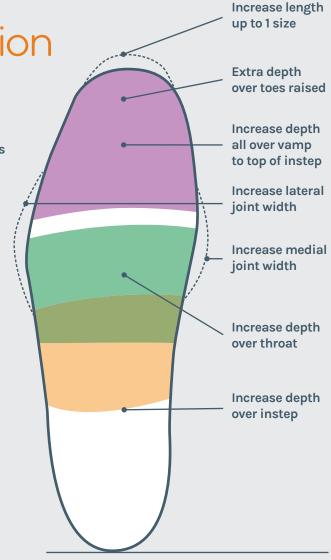

Measuring Guide

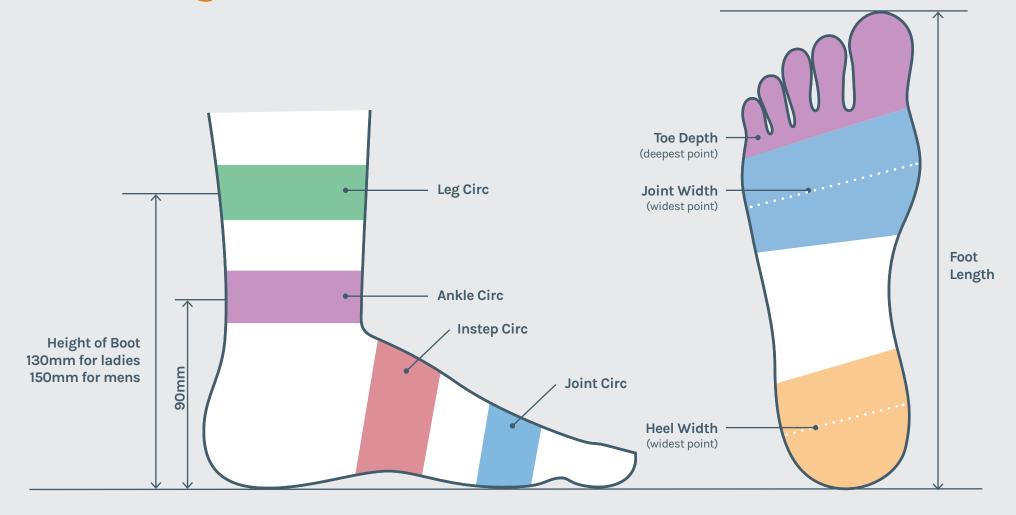

Measuring Guide

Notes on the size charts:

- As with standard practice, an allowance of 2.5
 English shoe sizes should be added to the draw size to get the shoe size required
- + Where a range of numbers are given, the largest number refers to the volume in the shoe with all insoles removed, the smaller number refers to the volume in the shoe with 9mm of insoles inside
- + Deep & wide and deep fitting is deeper at the toe box than standard or wide fittings, not at the instep
- + Deep & wide and deep fittings do not come in A. B or C lasts
- + A lasts are narrower at the heel by 4mm
- + B lasts are shallower at the forefoot by 4mm
- + C lasts are narrower at the heel by 4mm and shallower at the forefoot by 4mm
- + Medial collar height will be 8mm lower than back collar height
- + Lateral collar height will be 12mm lower than back collar height

Tips:

- + It is simpler to change from one fitting to another, than it is to make a last adaptation
- + For every 3mm of depth required, the circumference increases by 6mm
- At fitting, the guide on last and upper modifications remains. The total increase, including the initial modular increase, should not exceed 12mm over one point



Measuring Guide

These illustrations show the different variations on lasts



А

A lasts are narrower at the heel



STANDARD

Standard lasts have no modifications



WIDE

Wide lasts are wider along full-length, heel measure is wider than standard



В

B lasts are shallower at the forefoot – forefoot being from the joint forwards



SLIM

Slim lasts are slimmer along full length, heel width measure is less



DEEP & WIDE

Deep & wide lasts are deeper on the forefoot only and wider along the full length



C


C lasts are narrower at the heel and shallower at the forefoot – forefoot being from the joint forwards



DEEP

Deep lasts are deep at the forefoot from the joint forwards

Heel Guide

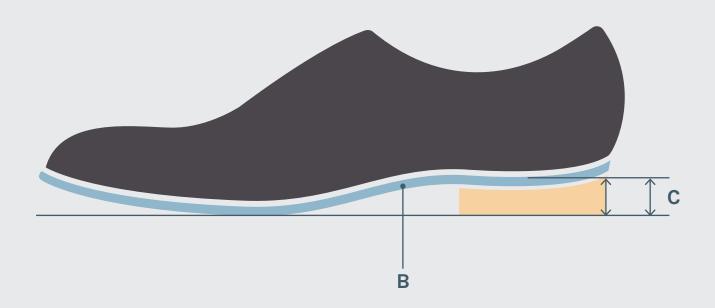

Key	
Sole Thickness	В
Heel Height	С

Ladies							
Measure	(mm)						
Sole Thickness	6						
Heel Height	25						

Mens								
Measure	(mm)							
Sole Thickness	8							
Heel Height	25							



LADIES - NARROW FITTING

Regul	Pegular Lasts						A Lasts - B Lasts - Shallow Forefoot			C Lasts – Narrow Heel & Shallow Forefoot				
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height	Heel Width (mm)	Instep Circ (mm)	Joint Circ (mm)	Toe Depth (mm)	Heel Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)
2	54	82	204 - 222	193 - 211	25 - 34	60	50	185 - 203	196 - 214	21 - 30	50	196 - 214	185 - 203	21 - 30
3	56	84	211 - 229	199 - 217	26 - 35	60	52	191 - 209	202 - 220	22 - 31	52	202 - 220	191 - 209	22 - 31
4	57	87	217 - 235	206 - 224	26 - 35	65	53	198 - 216	209 - 227	22 - 31	53	209 - 227	198 - 216	22 - 31
5	58	89	223 - 241	212 - 230	27 - 36	65	55	205 - 223	215 - 233	23 - 32	55	215 - 233	205 - 223	23 - 32
6	60	92	230 - 248	219 - 237	28 - 37	65	56	211 - 229	222 - 240	24 - 33	56	222 - 240	211 - 229	24 - 33
7	61	94	236 - 254	225 - 243	29 - 38	65	58	218 - 236	228 - 246	25 - 34	58	228 - 246	218 - 236	25 - 34
8	63	97	243 - 261	232 - 250	29 - 38	65	59	224 - 242	235 - 253	25 - 34	59	235 - 253	224 - 242	25 - 34
9	64	99	249 - 267	238 - 256	30 - 39	70	61	231 - 249	241 - 259	26 - 35	61	241 - 259	231 - 249	26 - 35

LADIES - STANDARD FITTING

Regul	egular Lasts									C Lasts – Narrow Heel & Shallow Forefoot				
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height	Heel Width (mm)	Instep Circ (mm)	Joint Circ (mm)	Toe Depth (mm)	Heel Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)
2	58	88	216 - 234	201 - 219	25 - 34	60	54	193 - 211	208 - 226	21 - 30	54	208 - 226	193 - 211	21 - 30
3	60	90	222 - 240	207 - 225	26 - 35	60	56	199 - 217	214 - 232	22 - 31	56	214 - 232	199 - 217	22 - 31
4	61	93	229 - 247	214 - 232	26 - 35	65	57	206 - 224	221 - 239	22 - 31	57	221 - 239	206 - 224	22 - 31
5	63	95	235 - 253	220 - 238	27 - 36	65	59	212 - 230	227 - 245	23 - 32	59	227 - 245	212 - 230	23 - 32
6	64	98	242 - 260	227 - 245	28 - 37	65	60	219 - 237	234 - 252	24 - 33	60	234 - 252	219 - 237	24 - 33
7	66	100	248 - 266	233 - 251	29 - 38	65	62	225 - 243	240 - 258	25 - 34	62	240 - 258	225 - 243	25 -34
8	67	103	255 - 273	240 - 258	29 - 38	65	63	232 - 250	247 - 265	25 - 34	63	247 - 265	232 - 250	25 - 34
9	69	105	261 - 279	246 - 264	30 - 39	70	65	238 - 256	253 - 271	26 - 35	65	253 - 271	238 - 256	26 - 35

LADIES - WIDE FITTING

Regul	Regular Lasts						A Lasts – Narrow Heel		B Lasts – Shallow Forefoot		C Lasts – Narrow Heel & Shallow Forefoot			
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height	Heel Width (mm)	Instep Circ (mm)	Joint Circ (mm)	Toe Depth (mm)	Heel Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)
2	62	94	227 - 245	209 - 227	25 - 34	60	58	201 - 219	220 - 238	21 - 30	58	220 - 238	201 - 219	21 - 30
3	64	96	234 - 252	215 - 233	26 - 35	60	60	207 - 225	226 - 244	22 - 31	60	226 - 244	207 - 225	22 - 31
4	65	99	240 - 258	222 - 240	26 - 35	65	61	214 - 232	233 - 251	22 - 31	61	233 - 251	214 - 232	22 - 31
5	67	101	247 - 265	228 - 246	27 - 36	65	63	220 - 238	239 - 257	23 - 32	63	239 - 257	220 - 238	23 - 32
6	68	104	254 - 272	235 - 253	28 - 37	65	64	227 - 245	246 - 264	24 - 33	64	246 - 264	227 - 245	24 - 33
7	70	106	260 - 278	241 - 259	29 - 38	65	66	233 - 251	252 - 270	25 - 34	66	252 - 270	233 - 251	25 - 34
8	71	109	267 - 285	248 - 266	29 - 38	65	67	240 - 258	259 - 277	25 - 34	67	259 - 277	240 - 258	25 - 34
9	73	111	273 - 291	254 - 272	30 - 39	70	69	246 - 264	265 - 283	26 - 35	69	265 - 283	246 -264	26 - 35

LADIES - DEEP FITTING

Regular Lasts											
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height					
2	58	88	227 - 245	213 - 231	31 - 40	60					
3	60	90	234 - 252	219 - 237	32 - 41	60					
4	61	93	240 - 258	226 - 244	32 - 41	65					
5	63	95	247 - 265	232 - 250	33 - 42	65					
6	64	98	254 - 272	239 - 257	34 - 43	65					
7	66	100	260 - 278	245 - 263	35 - 44	65					
8	67	103	267 - 285	252 - 270	35 - 44	65					
9	69	105	273 - 291	258 - 276	36 - 45	70					

LADIES - DEEP AND WIDE FITTING

Regular Lasts											
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height					
2	62	94	240 - 258	221 - 239	31 - 40	60					
3	64	96	246 - 264	227 - 245	32 - 41	60					
4	65	99	253 - 271	234 - 252	32 - 41	65					
5	67	101	259 - 277	240 - 258	33 - 42	65					
6	68	104	266 - 284	247 - 265	34 - 43	65					
7	70	106	272 - 290	253 - 271	35 - 44	65					
8	71	109	279 - 297	260 - 278	35 - 44	65					
9	73	111	285 - 303	266 - 284	36 - 45	70					

BOOTS

Regular Lasts				
Size	Leg Circ (mm) @130mm height			
2	196			
3	202			
4	209			
5	215			
6	222			
7	227			
8	234			
9	240			

MENS - NARROW FITTING

Regul	Regular Lasts						A Lasts - Narrow Heel		B Lasts – Shallow Forefoot		C Lasts – Narrow Heel & Shallow Forefoot			
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height	Heel Width (mm)	Instep Circ (mm)	Joint Circ (mm)	Toe Depth (mm)	Heel Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)
6	61	90	230 - 247	225 - 243	32 - 41	70	57	218 - 236	223 - 241	28 - 37	57	223 - 241	218 - 236	28 - 37
7	63	92	236 - 254	231 - 249	33 - 42	70	59	225 - 243	230 - 248	29 - 28	59	230 - 248	225 - 243	29 - 38
8	64	95	243 - 262	238 - 256	33 - 42	75	60	231 - 249	236 - 254	29 - 38	60	236 - 254	231 - 249	29 - 38
9	66	97	249 - 266	244 - 262	34 - 43	75	62	238 - 256	243 - 261	30 - 39	62	243 - 261	238 - 256	30 - 39
10	67	100	256 - 275	251 - 269	35 - 44	75	63	244 - 262	249 - 267	31 - 40	63	249 - 267	244 - 262	31 - 40
11	69	102	262 - 281	257 - 275	35 - 44	75	65	251 - 269	256 - 274	31 - 40	65	256 - 274	251 - 269	31 - 40
12	70	105	269 - 286	269 - 282	36 - 45	75	66	257 - 275	262 - 280	32 - 41	66	262 - 280	257 - 275	32 - 41

MENS - STANDARD FITTING

Regul	Regular Lasts						A Lasts - Narrow I		B Lasts – Shallow Fo	refoot	C Lasts – Narrow H	eel & Shallo	w Forefoot	
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height	Heel Width (mm)	Instep Circ (mm)	Joint Circ (mm)	Toe Depth (mm)	Heel Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)
6	65	96	241 - 260	233 - 251	32 - 41	70	61	226 - 244	235 - 253	28 - 37	61	235 - 253	226 - 244	28 - 37
7	67	98	247 - 266	239 - 257	33 - 42	70	63	233 - 251	242 - 260	29 - 38	63	242 - 260	233 - 251	29 - 38
8	68	101	254 - 273	246 - 264	33 - 42	75	64	239 - 257	248 - 266	29 - 38	64	248 - 266	239 - 257	29 - 38
9	70	103	260 - 279	252 - 270	34 - 43	75	66	246 - 264	255 - 273	30 - 39	66	255 - 273	246 - 264	30 - 39
10	71	106	267 - 286	259 - 277	35 - 44	75	67	252 - 270	261 - 279	31 - 40	67	261 - 279	252 - 270	31 - 40
11	73	108	273 - 292	265 - 283	35 - 44	75	69	259 - 277	268 - 286	31 - 40	69	268 - 286	259 - 277	31 - 40
12	74	111	280 - 299	272 - 290	36 - 45	75	70	265 - 283	274 - 292	32 - 41	70	274 - 292	265 - 283	32 - 41

MENS - WIDE FITTING

Regul	Regular Lasts						A Lasts - Narrow Heel		B Lasts – Shallow Forefoot		C Lasts – Narrow Heel & Shallow Forefoot			
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height	Heel Width (mm)	Instep Circ (mm)	Joint Circ (mm)	Toe Depth (mm)	Heel Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)
6	69	102	254 - 273	241 - 259	32 - 41	70	65	234 - 252	247 - 265	28 - 37	65	247 - 265	234 - 252	28 - 37
7	71	104	260 - 279	247 - 265	33 - 42	70	67	241 - 259	254 - 272	29 - 38	67	254 - 272	241 - 259	29 - 38
8	72	107	267 - 286	254 - 272	33 - 42	75	68	247 - 265	260 - 278	29 - 38	68	260 - 278	247 - 265	29 - 38
9	74	109	273 - 292	260 - 278	34 - 43	75	70	254 - 272	267 - 285	30 - 39	70	267 - 285	254 - 272	30 - 39
10	75	112	280 - 299	267 - 285	35 - 44	75	71	260 - 278	273 - 291	31 - 40	71	273 - 291	260 - 278	31 - 40
11	77	114	286 - 305	273 - 291	35 - 44	75	73	267 - 285	280 - 298	31 - 40	73	280 - 298	267 - 285	31 - 40
12	78	117	293 - 312	280 - 298	36 - 45	75	4	273 - 291	286 - 304	32 - 41	74	286 - 304	273 - 291	32 - 41

MENS - DEEP FITTING

Regu	lar Lasts	;				
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height
6	65	96	254 - 274	245 - 263	38 - 47	70
7	67	98	260 - 280	251 - 269	39 - 48	70
8	68	101	267 - 286	258 - 276	39 - 48	75
9	70	103	273 - 293	264 - 282	40 - 49	75
10	71	106	280 - 300	271 - 289	41 - 50	75
11	73	108	286 - 306	277 - 295	41 - 50	75
12	74	111	293 - 313	284 - 302	42 - 51	75

MENS - DEEP AND WIDE FITTING

Regu	lar Lasts	;				
Size	Heel Width (mm)	Joint Width (mm)	Joint Circ (mm)	Instep Circ (mm)	Toe Depth (mm)	Internal Back Collar Height
6	69	102	266 - 284	253 - 271	38 - 47	70
7	71	104	272 - 290	259 - 277	39 - 48	70
8	72	107	279 - 297	266 - 284	39 - 48	75
9	74	109	285 - 303	272 - 290	40 - 49	75
10	75	112	292 - 310	279 - 297	41 - 50	75
11	77	114	298 - 316	285 - 303	41 - 50	75
12	78	117	305 - 323	292 - 310	42 - 51	75

Regular Lasts Leg Circ (mm) Size @150mm height 223 228 8 235 9 241 10 248 254 12 260

BOOTS DURHAM

Regular Lasts						
Size	Leg Circ (mm) @100mm height					
6	242					
7	247					
8	254					
9	260					
10	267					
11	273					
12	279					



As some colours and styles have been discontinued, it will not always be possible to place an order for an

exact match to a legacy colour or style.

Costs for a legacy order will be greater than placing an order from the existing, updated stock and modular range. Please take careful consideration before making a legacy order and check first that the order can be placed.



Returns Policy

Options footwear manufactured on regular lasts and supplied in leather in black, willow or taupe and without alterations/adaptations or split size/fittings may be returned for a full refund.

Footwear manufactured on regular lasts and supplied in leather in any other colour, and without alterations and adaptations or split sizes/fittings, may be returned subject to a 25% handling charge.

Returns must be made within 90 days of receipt, in the original box and in their original condition. A returns authorisation number must be obtained from customer service before the footwear is returned.

Patient specific footwear, including split sizes, split fittings, non-standard leather (calf is standard), last or upper altered or adapted footwear are non-returnable.

Single shoes, boots and worn footwear are non-returnable.

Orders

We pride ourselves on offering first class customer service. Each customer is provided with their own Key Account Manager and Customer Service Representative and can call our dedicated helpline (Monday to Friday 8:00am – 5:00pm).

Orders should be submitted via phone, email, scan, fax or post. It will then be placed into our internal system, manufactured and posted via courier to the requested address.

Helpline 0191 276 9668

Email options@peacocks.net

Fax 0191 276 9655

Address

Peacocks Medical Group Benfield Business Park Benfield Road Newcastle upon Tyne NE6 4NQ



Peacocks Medical Group Ltd Benfield Business Park Benfield Road Newcastle upon Tyne NE6 4NQ

Customer Service: +44 (0) 191 276 9668

Email: options@peacocks.net Website: www.peacocks.net